

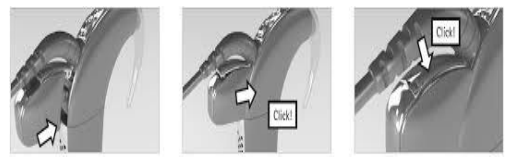



## Quick Set Up Sheet: Nucleus 6, Roger 14 Receiver, Touch Screen Mic/Transmitter and Soundfield System

Equipment	Directions
	<p>Roger Touch Screen Mic</p> <ol style="list-style-type: none"> <li>1) Charge it over night.</li> <li>2) Turn it on by pressing the side button for three seconds.</li> <li>3) The mic should be used on the lanyard within 4 inches of your mouth.</li> <li>4) Click the connect button in contact with Receivers and the Soundfield speaker.</li> <li>5) Mute the mic when not in use.</li> <li>6) Let the student have the mic on the desk if they are doing groupwork or working with their SNA.</li> </ol>
	<p>Roger Digimaster 5000</p> <ol style="list-style-type: none"> <li>1) Plug it in.</li> <li>2) Green light shows it is on, Blue light shows the mic is connected.</li> <li>3) Mute the mic when you are not speaking to the class, or turn the mic off.</li> </ol>
	<p>Roger 14 receiver</p> <ol style="list-style-type: none"> <li>1) Connect the receiver into the port, open the port drawer, push in receiver and close the drawer.</li> <li>2) Connect the mic to the receiver.</li> <li>3) A blue flashing light means the processor is paired to the mic.</li> <li>4) Use the monitoring earphones to hear what the student can hear.</li> <li>5) Report any issues to the VT.</li> </ol>
	<p>Roger Pass Around Mic</p> <ol style="list-style-type: none"> <li>1) Use the mic to allow the Deaf/HoH student to hear what the other students say.</li> <li>2) Sounds will be broadcast through the soundfield speaker.</li> <li>3) Connect the pass mic to the touch screen mic to use it.</li> </ol>
See user guides for further information	