





Quick Set Up Sheet: Nucleus 7, Roger 20 Receiver, Touch Screen Mic/Transmitter and Soundfield System

Equipment	Directions
	<p>Roger Touch Screen Mic</p> <ol style="list-style-type: none"> 1) Charge it over night. 2) To turn on push the side button for three seconds. 3) The mic should be used on the lanyard as close to your mouth as is comfortable. 4) Click the connect button in close contact with receivers and the soundfield speaker. 5) Mute the mic when not in use/turn it off. 6) Let the student have the mic on the desk if they are working with their SNA/ in groups. 7) Mic must be verticle if using passaround mic.
	<p>Roger Digimaster 5000</p> <ol style="list-style-type: none"> 1) Plug it in. 2) Green light shows it is on, Blue light shows the Mic is connected. 3) Mute or turn off the mic if you will not be talking for a few minutes.
	<p>Roger 20 reciever</p> <ol style="list-style-type: none"> 1) Connect the receiver in the slot between the processor and the battery. 2) Connect the mic to the receiver. 3) A blue flashing light means the processor is paired to the mic. 4) Use the listening headphones to hear what the student can hear. 5) Report problems to your VT.
	<p>Roger pass around Mic</p> <ol style="list-style-type: none"> 1) Use the mic to allow the Deaf/Hard of Hearing student to hear what the other students say. 2) It will broadcast to the soundfield speaker. 3) Connect the pass around mic to the touch screen mic to use it. 4) Teacher's mic must be verticle when the pass around mic is in use.
See user guide for more details	