



## How to check Nucleus 7 Processor and Receiver with Monitoring Earphones

### Parts of the Nucleus 7

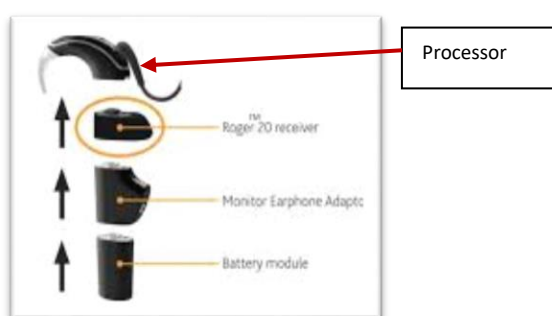

Remove the receiver from the Nucleus 7 processor.

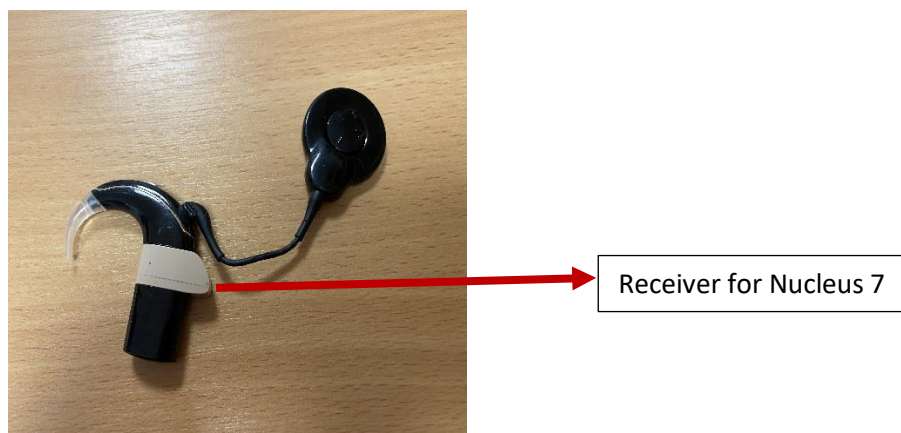

Attach monitoring earphone adaptor between the battery case and the processor. You should now be able to listen clearly through the processor to ensure it is working properly.



Monitoring Earphone Adaptor

Attach the receiver between the monitoring earphone adaptor and the processor.

Connect the receiver to the Roger Touchscreen microphone/transmitter and move 10 feet from the processor. Ask someone to talk into the Roger microphone. You should be able to hear them clearly through the monitoring earphones. If there is a Digimaster soundfield system in the room, ensure that it is switched off during this testing.