


How to Connect the Roger Select Microphone/Transmitter

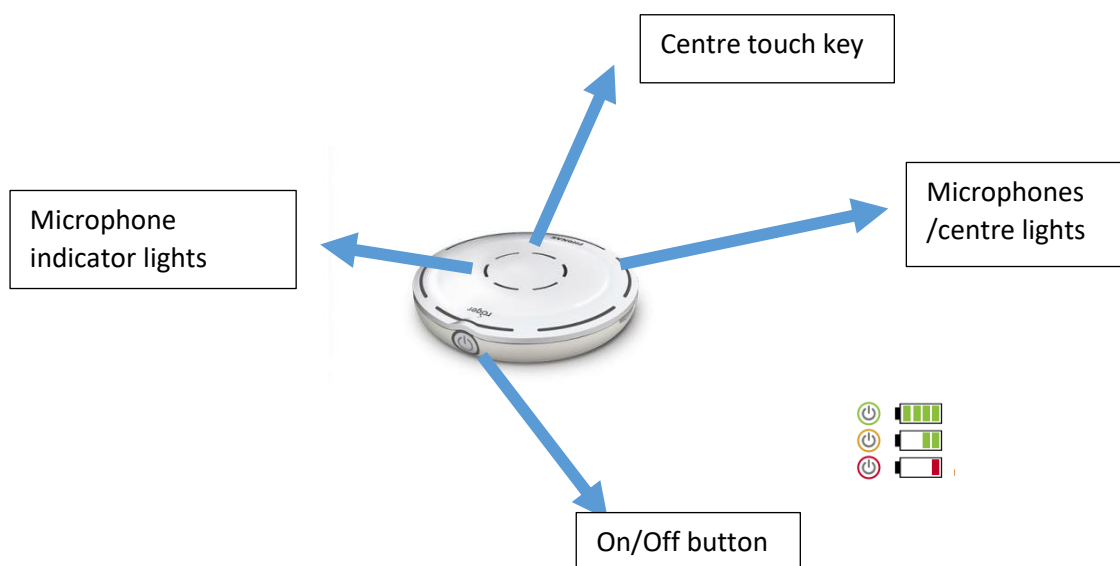

The Roger Select **must be fully charged** before use. Charge for three hours minimum.


- To turn on the microphone, press on/off button, **hold for three seconds**
- When the microphone is switched on, the status light around the on/off button will blink
- A green blinking light means battery is fully charged
- An orange blinking light means battery is half full
- A red blinking light means battery empty
- A blue blinking light means the device is connecting

Connecting the microphone to your receivers:

The Roger receiver **must be attached** to the hearing aid or cochlear processor

- Turn on the microphone
- Place the Roger microphone within 4cm from the Roger receiver.
- Press the connect icon/ button on the **back of the Roger microphone**. Press once and let go. The **back** indicator light will turn green for two seconds to indicate the connection was successful.

Controls on the back of the Roger Select

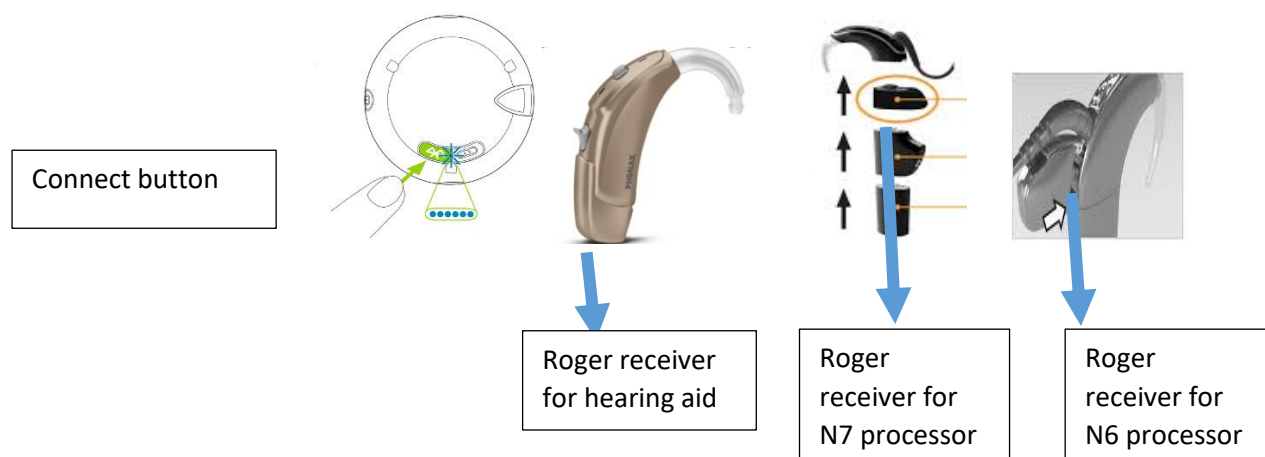

What different coloured lights mean

•••••	Connecting
•	Connection was successful – your Roger receiver is connected to your Roger microphone.
••	Your Roger microphone could not find a Roger receiver. Move the devices closer to your Roger receiver and make sure the Roger receiver is powered on. Then try to connect again.
•••	The Roger receiver is not compatible with your Roger microphone.

Blue= connecting

Green= connection to receiver of hearing aid/ cochlear processor successful

Red= connection to receiver of hearing aid/cochlear processor not successful

Wearing the Microphone

The Roger Select offers a choice of magnetic clip or lanyard for the wearer

- Attach the magnetic clip to the Roger Select microphone. Make sure it clips into the small slots in the side of the Roger microphone. Test strength of connection by pulling the Select and the clip apart. Ensure they are secure to prevent the microphone falling out of clip
- Open the magnetic clip
- Clip to the speaker's shirt/jumper
- Alternatively, use the lanyard provided. Attach it to the microphone by clicking it into small slots at the side
- The lanyard with microphone attached is worn around the neck of the speaker

The Roger microphone should be placed as close to the speaker's mouth as is comfortable (approximately 4 inches away). Adjust length of the lanyard as needed.

To test the equipment is working:

- Ask the pupil if they are happy with the sound quality with the microphone connected
- Ask the pupil to repeat a list of words/Ling sounds
- Alternatively, if you wish to test the equipment yourself, connect to phone and play music

If the pupil cannot respond correctly, repeat connection process.

Wearing the Microphone



Connecting the Microphone to a Laptop

- Insert the jack end of the audio cable into the port with headphone icon on the laptop
- Adjust the volume by changing volume settings on the laptop (speaker icon)
- Audio from SKYPE/YouTube video calls or PowerPoints can be directly streamed to the hearing aids or Cochlear Implants. This provides better clarity for the listener

